

PROPER MAINTENANCE AND UPKEEP WILL HELP PROLONG THE LIFE OF YOUR RAMP

Safety is our #1 priority. We encourage you to receive assistance from a qualified helper when necessary. Always adhere to your mobility device's operating guidelines. Caution should be used at all times. It is important that you are able to safely operate your mobility device with the degree of incline/decline and chair direction before attempting ramp use. Never exceed the safety recommendations from your installer.

DAILY UPKEEP

It is vital to keep all usable surfaces free & clear of dirt, leaves, snow/ice and any other accumulating debris that can impede the safety of the ramp system. Do not attempt to use your ramp if it is covered in leaves, dirt, debris, snow, and/or ice. It is critical that the non-skid face of the ramp system remain free of buildup to ensure traction remains fully intact. Remember, even non-skid treads can become slippery when wet or in wet condition. Please use extra caution on a damp/wet surface.

Please refrain from using any harsh chemicals and/or alkaline chemicals.

LEAF, DIRT, AND DEBRIS REMOVAL

Most debris can be removed by sweeping the ramp's surface with a broom and/or spraying it with a garden hose. For buildup on the surface that cannot be easily swept or rinsed away, a damp cloth and/or soft brush with a mild soap & water solution can be used.

Please refrain from using any harsh chemicals and/or alkaline chemicals as these can damage the chemical compounds of the metal, jeopardizing the structural integrity of the ramp system.

SNOW AND ICE REMOVAL

Snow should be removed with a shovel followed by sweeping the tread until clean & clear. If ice has built up on the ramp, a Magnesium Chloride flake or pellet is recommended to melt the ice. Once the snow/ice has been cleared, it is important to clean the ramp system with a mild soap & water solution.

Please refrain from using traditional rock salts as these can damage the welds and chemical compounds of the ramp, jeopardizing the structural integrity of the system.

ROUTINE MAINTENANCE

To maintain the safety of your ramp & its components, it is recommended that the ramp is frequently inspected for damage and to tighten loose fasteners. Should you find that any of the ramp is damaged or loose, do not attempt to repair this yourself - contact a certified LiftSquad dealer for assistance.